Instruction Manual

Fingertip Pulse Oximeter

Item # 40-810-000

Item # 40-811-000

Item # 40-812-000

Item # 40-813-000

Please read this guidebook
completely before operating this unit.

**DETAILED
GUIDEBOOK**

English • Spanish

Limited Two-Year Warranty

Your pulse oximeter is guaranteed to be free of manufacturing defects for a period of two years from the date of purchase under normal use. This limited warranty does not apply to use in any clinical or commercial applications. If the unit fails to operate during the warranty period, please call our Customer Care Help Line at 800-622-4714 and we will try to resolve the issue as soon as possible. If the warrantor determines the unit failed to operate due to a manufacturing defect, the unit will be replaced at the option of the warrantor. A return shipping and handling fee may be applied for all returns. Replacement of the unit is the sole remedy under this limited warranty. This warranty gives you specific legal rights, which vary from state to state. As a condition of this warranty, the warranty registration must be completed within 10 days of the purchase date at www.mabisdmi.com.

This warranty constitutes the warrantor's only responsibility and obligation to replace materials or components. We make no other express or implied warranties, arising by operation of law or otherwise, or any warranty of merchantability or fitness for a particular use or purpose whether or not the use or purpose has been disclosed to the warrantor in specifications, drawings or otherwise, and whether or not the warrantor's products are specifically designed and/or manufactured by the warrantor for the buyer's use or purposes, except for the limited warranty stated above. The warrantor will not be responsible for any indirect, incidental, special, consequential, or punitive damages or other loss, including, but not limited to, damage to or loss of other property or equipment and personal injuries, whether to purchaser or others. The warrantor shall in no event be liable to the purchaser for any amount in excess of the cost of replacement of the unit.

IMPORTANT SAFEGUARDS

Note: Read all instructions carefully before use.

Caution: Failure to read and observe all precautions could result in personal injury or equipment damage.

Index

Product Identification, page 4

Product Function Identification, page 5

Operating Overview, pages 6-7

Battery Installation, page 7

Important Information Before Use, page 8

Using Your Pulse Oximeter, page 9

Cleaning and Maintenance, page 10

Description of Display Symbols, page 10


Troubleshooting, page 11

Specifications, pages 11-12


Your Fingertip Oximeter is intended for measuring the pulse rate and functional oxygen saturation (SpO₂) through the finger. It is applicable for spot-checking of SpO₂ and pulse rates in homes.

This fingertip pulse oximeter is not a medical device and can not be used for medical diagnostic and treatment activities.


Product Identification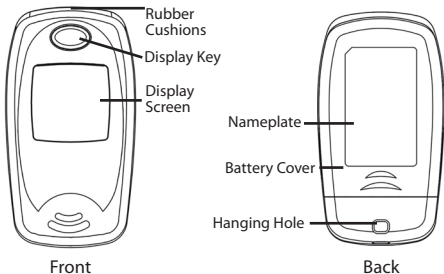

Measuring screen for Standard, 40-810-000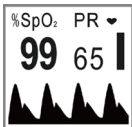

Measuring screen for Select, 40-811-000
Deluxe, 40-812-000
Premium, 40-813-000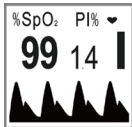

SpO₂ and PI screen for Select, 40-811-000
Deluxe, 40-812-000
Premium, 40-813-000

Product Function Identification

Model Function	40-810/ Standard	40-811/ Select	40-812/ Deluxe	40-813/ Premium
Display	LCD	OLED	OLED	OLED
Color	None	Monochrome	2 color	Full color
Rotation	1 Direction	2 Direction	2 Direction	4 Direction
SpO2	X	X	X	X
PR	X	X	X	X
PI		X	X	X
Pulse Bar	X	X	X	X
Plethys- ogram		X	X	X
Anti-Motion Technology		X	X	X
Auto Finger Detection		X	X	X
Auto On		X	X	X
Auto Off	X	X	X	X
Auto Rotation				X

Operating Overview

Operating Cautions

- Check the device to make sure there is no visible damage that may affect the user's safety or measurement performance with regard to sensors and clips. It is recommended that the device should be inspected minimally once a week. When there is obvious damage, stop using the device.
- Necessary maintenance must be performed by qualified service technicians. Users are not permitted to maintain it by themselves.
- The oximeter cannot be used together with devices not specified in this Instruction Manual.

Cautions

- Explosive hazard – DO NOT use this oximeter in an environment with inflammable gas such as some ignitable anesthetic agents.
- DO NOT use the oximeter while the testee is under MRI or CT scanning.

Warnings

- An uncomfortable or painful feeling may appear if using the oximeter continuously on the same place for a long time, especially for poor micro circulation patients. It is recommended that the oximeter should not be applied to the same location for longer than two hours. If any abnormal condition is found, please change the position of oximeter.
- DO NOT clip this device on edema or tender tissue.
- The light (infrared light is invisible) emitted from the device is harmful to the eyes, so service technician or testee should not stare at the light.
- The local law must be followed when disposing of the device.


Attention

- Keep the oximeter away from dust, vibration, corrosive substances, explosive materials, high temperature and moisture.
- The device should be kept out of the reach of children.
- If the oximeter gets wet, please stop using it and do not resume operation until it is dry. When it is carried from a cold environment to a warm and humid environment, please do not use it immediately.
- DO NOT operate the button on the front panel with sharp materials.
- DO NOT use high temperature or high pressure steam disinfection on the oximeter. Refer to page 10 for instructions of cleaning and disinfection.
- DO NOT use if the sensor is damaged.

Battery Installation

1. Slide cover off of backside of pulse oximeter.
2. Insert two "AAA" batteries into the compartment, matching the indicated polarity symbols.
3. To close, slide cover back into place.

It is recommended to remove the batteries if the unit will not be used for an extended period of time.

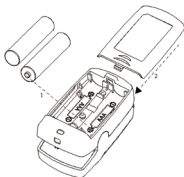Important Information Before Use

- The finger should be put in properly and correctly.
- Do not shake the finger. Keep at ease during measurement.
- Avoid placing the device on the same limb which is wrapped with a cuff for blood pressure measurement during venous infusion.
- Do not let anything block the emitting light from device.
- Vigorous exercise and electrosurgical device interference may affect the measuring accuracy.
- The gravity sensor works on the basis of gravity. A small movable metal ball is built in the gravity-sensor for detecting the orientation of the oximeter. When you want to change the oximeter's display direction, if you move the oximeter too slowly, the movable ball will also move slowly because of not enough acceleration. Consequently the response of orientation detection would be delayed. Please move the oximeter with a bit of force if you want to change the display direction (such as bend/extend your finger quickly), so an acceleration is provided to the gravity-sensor for quick sensing the orientation change.
- Due to the working principle of orientation sensor used in the oximeter, there is a small metal ball which is movable within its compartment of the gravity sensor. Therefore you may hear a slight "clatter" sound when you wave or shake the oximeter. It is normal and not caused by an unwanted part.
- Using enamel or other makeup on the nail may affect the accuracy of the measurement.

Using Your Pulse Oximeter

Standard, 40-810-000

1. Open the clip and place finger into the rubber cushions of the clip, and clip shut.
2. Press the display key to begin measurement.
3. Remove finger and the oximeter will power off automatically.

Select, 40-811-000 and Deluxe, 40-812-000

1. Open the clip and place finger into the rubber cushions of the clip, and clip shut.
2. Wait two seconds and the oximeter will power on automatically and begin measurement.
3. Press and release the Display Key to change the display direction.
4. Press and hold the Display Key to shift the parameter display between PR and PI.
5. Remove finger and the oximeter will power off automatically.

Premium, 40-813-000

1. Open the clip and place finger into the rubber cushions of the clip, and clip shut.
2. Wait two seconds and the oximeter will power on automatically and begin measurement. Tilt the oximeter to change display direction.
3. Press and hold the Display Key to shift the parameter display between PR and PI.
4. Remove finger and the oximeter will power off automatically.

Cleaning & Maintenance

Cleaning

- Surface-clean sensor with a soft cloth by wetting with a solution, such as 75% isopropyl alcohol. If low-level disinfection is required, use a 1:10 bleach solution.
- Surface-clean body with a cloth saturated with clean water and dry with a clean, soft cloth.

Caution: Do not sterilize by irradiation steam or ethylene oxide.

Maintenance

In order to ensure long service life for your pulse oximeter, please follow the below maintenance guidelines.

- Change the batteries when the low indicator light appears.
- Clean the surface of the device before using.

Caution:

- High-pressure sterilization cannot be used on this device.
- Do not immerse device in liquid.
- It is recommended that the device should be kept in a dry environment. Humidity may reduce the life of the device, or even damage it.

Description of Display Symbols

	Pulse oxygen saturation
 BPM/PR	Pulse rate (beats per minute)
PI%	Perfusion Index (%)
	Pulse intensity bar graph
	Low battery indicator

Troubleshooting

Display direction does not change or changes slowly.	Shake the oximeter with minimal force to make the movable metal ball move freely.
The SpO2 and pulse rate are unstable.	Make sure the finger is inserted correctly. Review the "Important Information Before Use" section.
The device will not turn on.	Check and correct the polarity of the installed batteries. Reinstall or replace batteries.
Display is blank.	Check and correct the polarity of the installed batteries. Reinstall or replace batteries.

Specifications

Name and Item No.	Standard Pulse Oximeter, 40-810-000 Select Pulse Oximeter, 40-811-000 Deluxe Pulse Oximeter, 40-812-000 Premium Pulse Oximeter, 40-813-000
Power Source	2 "AAA" batteries (1.5V) Supply voltage: 3VDC Operating current: $\leq 40\text{mA}$
SpO2 Parameter	Measuring range: 35% - 99% Accuracy: 70% - 100% $\pm 3\%$

Specifications

Pulse Rate Parameter	Measuring range: 30bpm - 240 bpm Accuracy: ± 2 bpm or $\pm 2\%$ (whichever is greater)
Perfusion Index (PI) Display	Range: 0% - 20%
Operation Environment	Temperature: 41°F - 104°F (5°C - 40°C) Humidity: 30% - 80% Pressure: 70kPa - 106kPa
Performance Under Low Perfusion Condition	Same with modulation amplitude as low as 0.6%
Resistance to Light Interference	$\pm 1\%$
Resistance to 50Hz/60Hz Interference	Precise
Unit Dimensions	2.6" L x 1.4"W x 1.3"H
Unit Weight	2 oz.
Accessories	Lanyard, two "AAA" batteries, pouch, detailed guidebook

Specifications are subject to change without notice.

Manufactured for:
Briggs Medical Service Company
1931 Norman Drive
Waukegan, IL 60085
www.mabisdmi.com

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© 2011 Briggs Medical