# *Instruction Manual*

---

## *Automatic Digital Blood Pressure Wrist Monitor with Dual Memory Banks*

Model 04-243-001

Please read this instruction manual  
completely before operating this unit.

# Health ♥ Smart **R<sub>x</sub>**

**DETAILED  
GUIDEBOOK**

*English • Spanish*

# Limited One-Year Warranty

---

The warrantor guarantees that its Digital Blood Pressure Monitor will be free from manufacturing defects under normal use for a period of one year from the date of purchase subject to the following terms and conditions.

This warranty covers only normal use, and does not apply to use in any clinical or commercial applications. This warranty does not cover batteries or other power sources that may be provided with or used with the Digital Blood Pressure Monitor. This warranty is voided if the Digital Blood Pressure Monitor product is misused or abused in any manner.

If the Digital Blood Pressure Monitor fails to operate during the time the original purchaser owns it, return it postage prepaid along with \$5.00 for return shipping and insurance to: Repair Department, 1931 Norman Drive, Waukegan, IL 60085. When returning a product, please also include a letter with your name, address, phone number and brief description of the specific problem. Please carefully package the product to avoid any damage that may occur while in the transit. Packaging insurance with returned receipt is recommended. The warrantor will repair or replace the defective unit at the option of the warrantor. Repair or replacement of the defective unit, at the warrantor's option, is the sole remedy under this warranty.

ANY IMPLIED WARRANTIES WHICH THE PURCHASER MAY HAVE ARE LIMITED IN DURATION TO THE TIME THAT THE ORIGINAL CONSUMER PURCHASER OWNS THE PRODUCT. Some states do not allow limitations or how long an implied warranty lasts, so the above limitation may not apply to you.

This warranty constitutes the warrantor's only responsibility and obligation to repair and/or replace materials or components, or refund the purchase price. **The warrantor will not be responsible for any indirect, incidental, special, consequential, or punitive damages or other loss**, including, but not limited to, damage to or loss of other property or equipment and personal injuries, whether to purchaser or others. MABIS shall in no event be liable to the purchaser for any amount in excess of the cost of repair and/or replacement of the unit, or the purchase price of the unit. Some states do not allow the exclusive or limitation of incidental or consequential damages, so the above limitation or exclusion may not apply to you.

This warranty gives you specific legal rights, and you may also have other rights which vary from state to state.

Toll-Free Information Line:

1-800-622-4714

Monday - Friday 8:00 am - 4:30 pm CST

## HealthSmart RX

1931 Norman Drive • Waukegan, IL 60085

847-680-6811 • FAX 847-680-9646

# Index

---

Introduction .....	2
Product Identification and Caution .....	3
General Blood Pressure Information .....	4-6
Important Information Before Use .....	7
Battery Installation/Replacement .....	8
Setting the Date/Time .....	9-11
Applying Your Blood Pressure Cuff .....	12-13
Taking Your Blood Pressure Reading .....	14-15
Recalling Measurements in Memory .....	16
Description of Display Symbols .....	17
Troubleshooting .....	18
Care and Maintenance .....	18
Product Specifications .....	19
Sample Blood Pressure Log .....	20
Blood Pressure Log .....	21

# Introduction

---

To achieve the maximum benefit from your blood pressure monitor, we recommend that you first consult with your physician or trained health care professional.

Thank you for purchasing an Automatic Blood Pressure Monitor. With proper care and use, your monitor will provide you with many years of reliable readings.

The method of measurement that your Automatic Monitor uses is called the oscillometric method. The monitor detects your blood's movement through the artery in your wrist and converts the movements into a digital reading. The oscillometric method does not require a stethoscope, making the monitor easy-to-use. The preformed cuff requires little effort to apply and is comfortable to wear.


Clinical research has proven a direct relationship between blood pressure in the wrist and blood pressure in the arm. Because the arteries in the wrist and arm are connected, changes in wrist blood pressure reflect changes in arm blood pressure. Frequent wrist blood pressure measurements will provide you and your physician with an accurate indication of change in your true blood pressure.

**People with severe hypertension, severe arteriosclerosis, or severe diabetes, should consult their physician about monitoring blood pressure at the wrist.**

*Blood pressure readings determined with this device are equivalent to measurements obtained by a trained observer using the cuff/stethoscope auscultation method, within the limits prescribed by the American National Standard for Electronic or Automated Sphygmomanometers.*

# Product Identification and Caution

---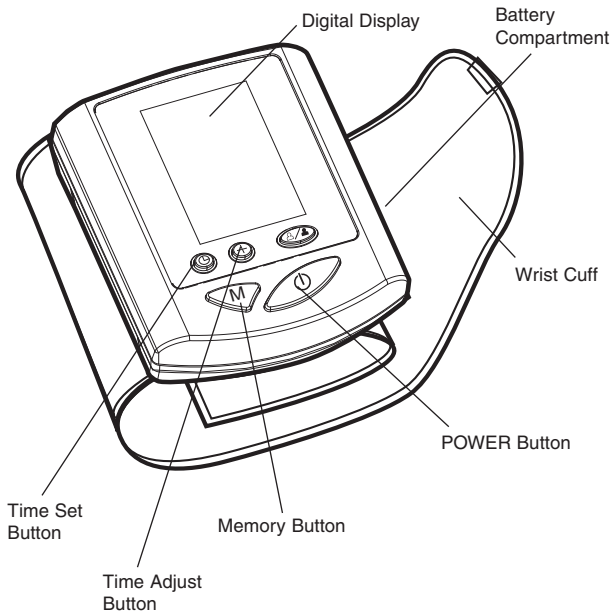

## Caution

While taking a measurement, you can stop the inflation or deflation process of the cuff at any time by pressing the POWER button.

# General Blood Pressure Information

## **What is Blood Pressure?**

Blood pressure is the pressure that is exerted by blood flowing against the walls of the blood vessels throughout your body.

Your heart, which is the center of the circulatory system, provides the force for the blood to flow or circulate. When your heart contracts or beats, the blood is forced through the blood vessels increasing the pressure. This is the highest pressure in the cycle or what is referred to as **SYSTOLIC BLOOD PRESSURE**. In between beats, your heart relaxes and your blood pressure decreases. This is referred to as **DIASTOLIC BLOOD PRESSURE**.

This complete series of events which occurs in a single heartbeat is known as the **CARDIAC CYCLE**.

Your Digital Blood Pressure Monitor will automatically read your blood pressure and display both systolic and diastolic readings on the screen. Your systolic will be positioned as the upper number and the diastolic reading will be the lower number.

(systolic) **120/80** (diastolic)

Blood pressure is measured in millimeters (mm) of Mercury (Hg) and is generally recorded with the systolic pressure (120) listed first and the diastolic pressure (80) listed second. The numbers are typically separated by a slash mark (/) as shown above.

*Both pressure readings, the **SYSTOLIC** and the **DIASTOLIC**, are necessary for a physician to evaluate the status of a patient's blood pressure.*

*Please contact your physician for specific information regarding your own blood pressure.*

## **What Influences Blood Pressure?**

Many factors such as genetics, age, sex, altitude, physical activity, anxiety, muscular development, certain medications or even the time of day can influence blood pressure. Influences such as sleep or relaxation decrease blood pressure, while anxiety or exercise increases blood pressure.

## **Why Monitor Your Blood Pressure at Home?**

A visit to a physician's office can be a stressful situation for a patient. And, anxiety is a known factor in raising blood pressure. This temporary occurrence of elevated blood pressure at the physician's office is commonly referred to as "white coat syndrome".

Whether or not you experience "white coat syndrome", home blood pressure monitoring provides you with the opportunity to supplement your physician's office measurements. These home readings, when taken over a period of time, can show an accurate indication of change. Furthermore, your records can assist your physician in evaluating your health and in making important decisions in the diagnosis and treatment of your condition. Because of this, it is important to take consistent, daily measurements of your blood pressure.

*The variations in your individual readings should only be interpreted by your physician or a trained health care professional.*

## **WHO Blood Pressure Classifications**


Standards for assessment of high or low blood pressure, regardless of age, have been established by the World Health Organization (WHO) as shown in the chart on the following page.

CATEGORY	SYSTOLIC (mmHg)	DIASTOLIC (mmHg)
Optimal	<120	<80
Normal	<130	<85
High - Normal	130-139	85-89
Mild Hypertension	140-159	90-99
Borderline	140-149	90-94
Moderate Hypertension	160-179	100-109
Severe Hypertension	≥180	≥110
Isolated Systolic Hypertension	≥140	<90
Borderline	140-149	<90

*This chart is only a general guideline. Please contact your physician to determine your **NORMAL BLOOD PRESSURE**.*

### **Variations in Blood Pressure**

Blood pressure can be influenced by many factors. This change can occur from moment to moment. Normally, blood pressure is lowest during the sleeping period and rises during the day. The graph below represents the variations in blood pressure shown over a day with measurements taken every 5 minutes.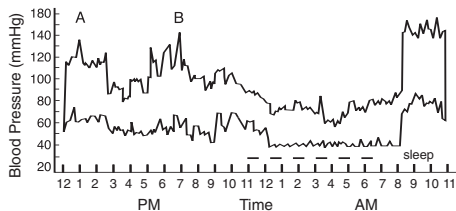

The dotted line represents the sleep period. The rise in blood pressure at 1 p.m. (A in the graph) corresponds to a stressful occurrence and at 7 p.m. (B in the graph) a period of exercise.

## ***Important Information Before Use***

1. Blood pressure measurements should only be interpreted by a physician or a trained health care professional who is familiar with your medical history. Through regular use of this device and recording of your measurements, you can keep your physician informed of the changes in your blood pressure.
2. Perform your measurement in a quiet place. You should be seated in a relaxed position.
3. Avoid smoking, eating, taking medication, alcohol consumption or physical activity 30 minutes prior to taking a reading. If you are exhibiting signs of stress, avoid taking your measurement until the feeling subsides.
4. Rest 15 minutes prior to taking a reading.
5. Remove any constrictive clothing or jewelry that may interfere with the cuff placement.
6. Apply the cuff to the proper position on the left wrist. The unit should be located on the inside of your left wrist with the display panel in view.
7. Keep the monitor stable during measurement to achieve an accurate reading. Remain still; do not talk during the measurement.
8. Record your daily blood pressure and pulse readings on a chart.
9. Take your readings at the same time, each day (or as recommended by your physician) to get an accurate indication of change in your true blood pressure.
10. Wait a minimum of 15 minutes between readings to allow for the blood vessels to return to normal. The wait time may vary depending on your individual physiological characteristics.
11. Although such cases are rare, for those with an extremely weak pulse or irregular pulse, errors may result which prevent proper measurement. If abnormal variations are noticed, consult with your physician or trained healthcare professional.
12. This device is intended for adult use.

*While taking a measurement, you can stop the inflation or deflation process of the cuff at any time by pressing the POWER button.*

# Battery Installation/Replacement

Replace the batteries when the Low Battery symbol appears on the display, *Fig. 1*, or when the display remains blank when the POWER Button is pressed.

To change the batteries:

1. Open the battery cover, *Fig. 2*.
2. Making sure the battery polarity is correct, install two AAA alkaline batteries, *Fig. 3*.
3. Close the battery cover, *Fig. 4*.

**NOTE:** It is recommended to remove the batteries if the unit will not be used for an extended period of time.

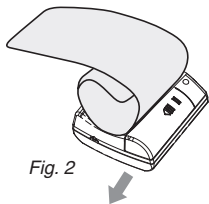
Fig. 2

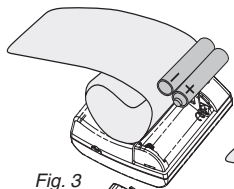
Fig. 3

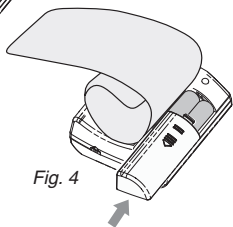
Fig. 4

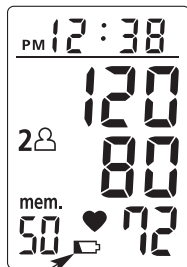
Fig. 1

# Setting the Date/Time

The two buttons positioned toward the bottom of the display are for setting the date and time, *Fig. A*.

Ⓢ **Time Set button** – Sets the entry and/or moves to the next mode.

⊕ **Time Adjust button** – Increases to the desired value.

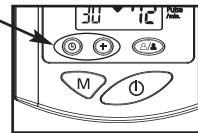
Fig. A

## NOTE:

- The appropriate mode must be flashing to make a change. Use the Ⓢ Time Set button to move within modes (Month/Date/Hour/Minute); otherwise, the mode will automatically change to the next mode after approximately 10 flashes.

## To enter into time/date mode:

1. Press and hold the Ⓢ Time Set button for approximately 3-5 seconds until the date/time appears. Release the button.

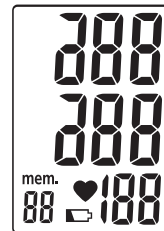
Fig. 1

## Setting the Date/Time

### To change the month:


2. With the month mode flashing, press the ⊕ Time Adjust button until the desired month appears, *Fig. 2*.

3. Press the ⊙ Time Set button to set the entry.

### To change the date:

4. With the date mode flashing, press the ⊕ Time Adjust button until the desired date appears, *Fig. 3*.

5. Press the ⊙ Time Set button to set the entry.

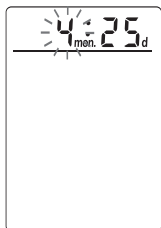*Fig. 2*

*Fig. 3*

## Setting the Date/Time

### To change the hour:


6. With the hour mode flashing, press the ⊕ Time Adjust button until the desired hour appears, *Fig. 4*.

7. Press the ⊙ Time Set button to set the entry.


### To change the minutes:

8. With the minute mode flashing, press the ⊕ Time Adjust button until the desired minutes appear, *Fig. 5*.

9. Press the ⊙ Time Set button to set the entry.

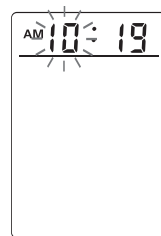*Fig. 4*

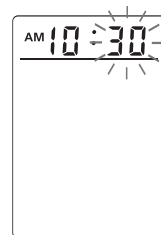
*Fig. 5*


# Applying Your Blood Pressure Cuff

Avoid smoking, eating, taking medication, alcohol consumption or physical activity 30 minutes prior to taking a reading. If you are exhibiting signs of stress, avoid taking your measurement until the feeling subsides. Rest 15 minutes prior to taking a reading.

1. Remove any constrictive clothing or jewelry that may interfere with the cuff placement.
2. You should be seated with your feet flat on the floor.
3. Hold your left arm in front of you with your palm facing upward.
4. Apply the preformed cuff to your left wrist. The digital display panel should be positioned on the inside area of your wrist facing you, *Fig. 1*.
5. Adjust the cuff as shown in *Fig. 2* from the edge of the 'Head of the Ulna Bone.'

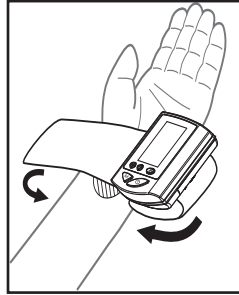
*Fig. 1*

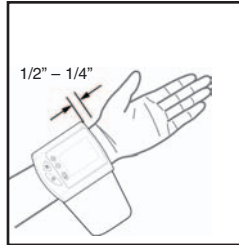*Fig. 2*

6. Wrap and fasten the cuff securely, yet comfortably around your wrist, *Fig. 3*.

**NOTE:** The left wrist should be used unless you are unable to or should not use your left arm or wrist. In consultation with your physician, you may modify the following instructions and apply the cuff to your right wrist.

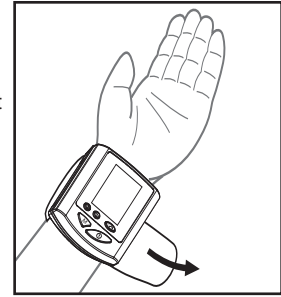*Fig. 3*

# Taking Your Blood Pressure Reading

Proceed only after reading the previous sections of this manual.

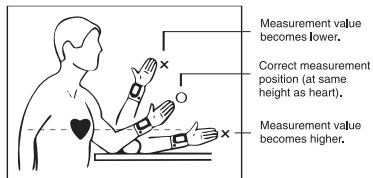
Fig. 1

1. Rest your elbow on a table with your palm facing upward. **Raise your hand so that the cuff is at the same level as your heart**, Fig. 1. Relax your hand.
2. Select the memory bank you want your reading to be stored in by pressing the User-Select button to alternate between memory bank 1 or 2, Fig. 2. Press the POWER button to confirm your selection.
3. Press the POWER button. This will turn the power on, Fig. 3.
4. When the display appears as shown, Fig. 4, the monitor is ready.

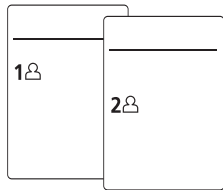
Fig. 2

Fig. 3

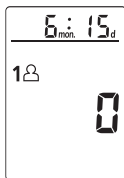
Fig. 4

# Taking Your Blood Pressure Reading

Fig. 5

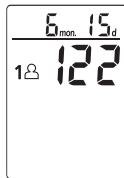
Fig. 6

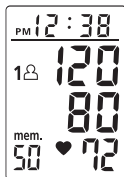
Fig. 7

5. The cuff will automatically inflate to approximately 180 mmHg.
6. Once the cuff inflates to approximately 180 mmHg, measurement will begin, Fig. 5.
7. The unit will begin to automatically deflate. The numbers appearing on the display will decrease, Fig. 6.
8. When all air is deflated, the measurement is complete. The unit will display your blood pressure (systolic and diastolic) measurements and pulse rate, Fig. 7.
9. Record your readings on a chart.
10. At this point, your measurement will be automatically saved in memory. Up to 100 measurements can be saved into two separate memory banks (50 each). Your unit will automatically shut off after approximately 1 minute.

*This monitor inflates the cuff to approximately 180mmHg. If the system detects that the cuff needs additional pressure, it will automatically reinflate to approximately 220 mmHg.*

*If you need to take another measurement, wait a minimum of 15 minutes between readings to allow for the blood vessels to return to normal. The wait time may vary depending on your individual physiological characteristics.*

## Recalling Measurements in Memory

You can recall up to 100 total measurements in memory (50 per memory bank) plus an average of the last three measurements in memory. These measurements can be shared with your physician or trained health care professional.

1. Press the MEMORY button. The first reading displayed is the average of the last three measurements stored in memory, *Fig. 1*.
2. Continue to press the MEMORY button to view the next previously stored measurement, *Fig. 2*.

**NOTE:** Each memory bank can store up to 50 readings. When the number of readings exceeds 50, the oldest data will be replaced with the new record.

All stored measurements will be erased by removing and reinstalling one of the batteries.

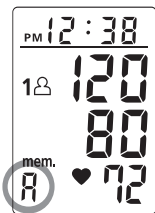
Fig. 1

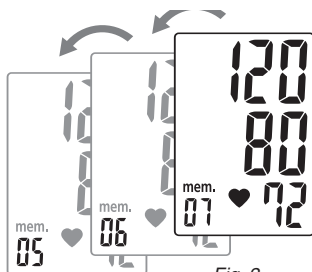
Fig. 2

## Description of Display Symbols

SYMBOL	CONDITION	ACTION
	Appears when pulse reading is displayed.	Record pulse reading.
mem.	Appears when recalling a measurement from memory.	Memory recall function in progress.
1  2 	Memory banks.	Select bank for memory storage or recall.
	Appears when battery voltage is excessively low.	Replace both batteries with new ones.
EE	Appears when an accurate reading cannot be obtained.	Wait a minimum of 15 minutes and remeasure. Loosen cuff to allow blood circulation in the wrist and hand while waiting.
E 1	Appears when cuff is over pressurized.	Wait a minimum of 15 minutes and remeasure. Loosen cuff to allow blood circulation in the wrist and hand while waiting.
E 2	Appears when there is too much movement during measurement.	Do not talk or move during measurement. Review 'Applying Your Blood Pressure Cuff' section.
E 3	Appears when unit has malfunctioned.	Unit needs to be sent in for repair or replacement.

## Troubleshooting

---

If any abnormality occurs during use, please check and correct the following:

<b>Condition</b>	<b>Correction</b>
Display is blank when power is on.	Check and correct the polarity of the installed batteries. Reinstall or replace batteries.
Measurement incomplete or abnormally low or high values displayed.	Review and follow 'Applying Your Blood Pressure Cuff' and 'Taking Your Blood Pressure Reading' sections. Replace the batteries.
Measurements are different from those typically measured by physician or every measurement reading is different.	Remember, blood pressure readings are influenced by physical and mental conditions and/or even the time of day. Daily measurement recordings should be interpreted by your physician.

## Care and Maintenance

---

1. Only use a soft, dry cloth to clean your blood pressure monitor.
2. Avoid using any types of liquids on the monitor or cuff.
3. Store your monitor in the provided storage case when not in use.
4. Do not store the unit where it will be exposed to direct sunlight, dust or humidity. Avoid extreme temperatures.
5. Never disassemble the monitor or cuff.
6. Dropping or subjecting your blood pressure monitor to strong shocks should be avoided.

## Product Specifications

---

<b>Name and Model Number</b>	HealthSmart RX Automatic Digital Blood Pressure Wrist Monitor with Dual Memory Banks, 04-243-001
<b>Display System</b>	Digital display/LCD
<b>Measuring Method</b>	Oscillometric
<b>Power Source</b>	2 AAA batteries
<b>Measuring Range</b>	Pressure: 40 to 250 mmHg Pulse: 40-199 beats/minute
<b>Accuracy</b>	Pressure: $\pm 3$ mmHg; Pulse: $\pm 5\%$ of reading
<b>Pressurization</b>	Automatic pressurization by pump
<b>Deflation</b>	Automatic pressure release valve
<b>Memory</b>	Two built-in memory banks enabling display of up to 50 measurements each, plus the average of the last three readings in memory
<b>Automatic Power Off</b>	Approximately 1 minute after last button operation
<b>Battery Life</b>	Approximately 200 measurements
<b>Operation</b>	
<b>Temperature</b>	50°F – 104°F (10°C – 40°C)
<b>Humidity</b>	40% to 85% RH (non-condensing)
<b>Storage</b>	
<b>Temperature</b>	14°F – 140°F (-10°C – 60°C)
<b>Humidity</b>	10% to 95% RH (non-condensing)
<b>Weight</b>	4.2 oz. (without batteries)
<b>Dimensions</b>	2-1/2" x 3-1/8" x 1-1/4"
<b>Wrist Circumference Range</b>	Suitable for 5-1/8" – 8-3/4" (13.5 – 22 cm)
<b>Accessories</b>	Storage case, Instruction Guidebook, two AAA alkaline batteries, Quick Start Guide

Specifications are subject to change without notice.

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©2006 HealthSmart RX